

Rad-57™

TRAINING TIPS



When using the RAD-57 to measure SpCO consider the following:

- If possible, confirm pre-incident SpCO level and record for reference
- Proper probe placement is important
- Finger should be relatively clean and warm for optimal reading
- Low levels of SpCO may be normal, typically less than 5
- If SpCO is 10 or above, reconfirm measurement by measuring several fingers (Do not use thumb or small finger/5th finger)
- If SpCO remains above 10, consider further evaluation and treatment as per local protocol
- Always evaluate the patients condition as the primary means of determining treatment
- Always follow your local EMS protocols and consult your local EMS medical director



Use the
2nd, 3rd and
4th fingers
for
measuring

SpCO
display will
show % of
CO and
letters
"CO"

